


How to order the GeneSight® test

Sample collected by patient at home

1

Enter the order at myGeneSight.com

Click the NEW ORDER button on the homepage

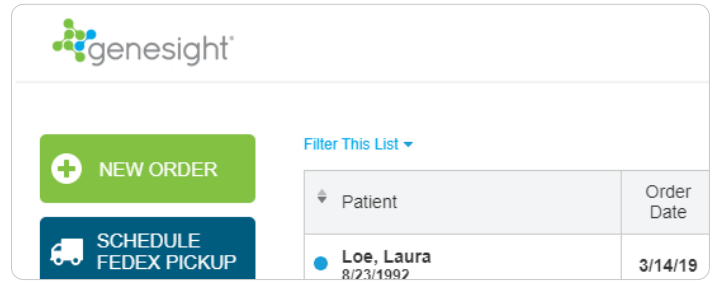

genesight®

NEW ORDER

SCHEDULE FEDEX PICKUP

Filter This List ▾


Patient	Order Date
Loe, Laura 8/23/1992	3/14/19

2

Ship kit to patient

In the Patient Information section, select Yes to Ship a Kit

- Confirm patient address is accurate
- We will ship a kit to your patient in 24-48 hrs
- We will notify your patient with status updates
- The sample status can be viewed on the homepage

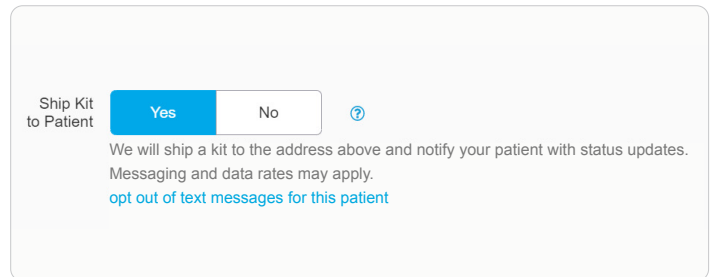

Ship Kit to Patient Yes No [?](#)

We will ship a kit to the address above and notify your patient with status updates. Messaging and data rates may apply.
[opt out of text messages for this patient](#)

3

Clinician or staff completes the order

If staff places a Medicare / Medicare Advantage order, clinician will need to authorize by:

Clinician must authorize order before patient collects sample

Manual Signature:

Clinician signs printed form

- upload to the order via myGeneSight.com, email to support@genesight.com, or fax to 888.894.4344

Electronic Signature:


Clinician signs via e-signature

- click 'Sign and Authorize Order' on myGeneSight.com

4

Patient collects samples using GeneSight Patient Collection Kit


Patients are asked to return samples within 2 days of receiving their kit

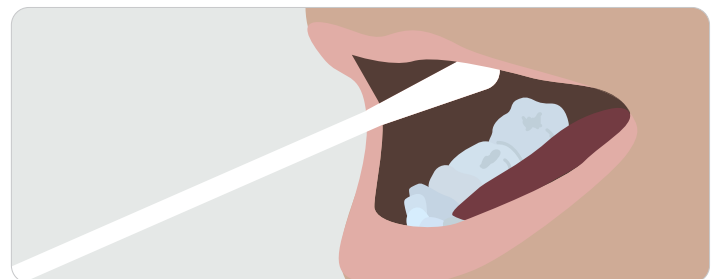

5

View your patient's GeneSight test results

Once the patient's samples are returned and their results are ready, you will receive an email notification to view them on myGeneSight.com

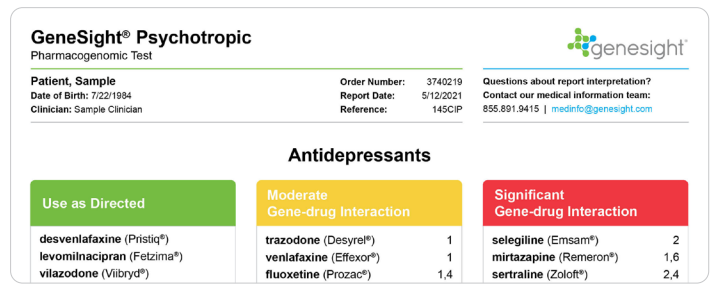

GeneSight® Psychotropic
Pharmacogenomic Test

Patient, Sample
Date of Birth: 7/22/1984
Clinician: Sample Clinician

Order Number: 3740219
Report Date: 5/12/2021
Reference: 145CIP

Questions about report interpretation?
Contact our medical information team:
855.891.9415 | medinfo@genesight.com

Antidepressants

Use as Directed	Moderate Gene-drug Interaction	Significant Gene-drug Interaction
desvenlafaxine (Pristiq®)	trazodone (Desyrel®)	selegiline (Emsam®)
levomilnacipran (Fetzima®)	venlafaxine (Effexor®)	mirtazapine (Remeron®)
vilazodone (Viibryd®)	fluoxetine (Prozac®)	sertraline (Zoloft®)

Customer Service: Phone: 866.757.9204 | support@genesight.com | Fax: 888.894.4344

© 2021 Assurex Health, Inc. doing business as Myriad Neuroscience.
GeneSight and associated logos are registered trademarks of Assurex Health, Inc. All Rights Reserved.

GSAP050.21Z

